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Used in cases of epicondylitis inflammations, traumas, injuries, and during the post-operative recovery period.
Depending on need and use, it helps healing by relieving the load on the tendons in the medial or lateral epicondyle.
Can also be used as a protective support during sports activities.

WARNING :

This product is designed for single patient use. Must not be exposed to sunlight, dust or humidity. The product is worn as in the photograph. It is used as per doctor's advice. Always consult a doctor. If rash develops, pain is prolonged or conditions worsen; consult a doctor. Avoid use the product, if you have sensitivity or allergy against the materials which the product contains.

WASHING PROCESS :

It is washed without rubbing at 30 degrees of water with bar soap by hand. Do not use a washing machine. It is dried at the room temperature on a towel. Do not tumble dry, do not dry clean, do not wring, do not iron, do not bleach, do not use detergent and fabric softener.

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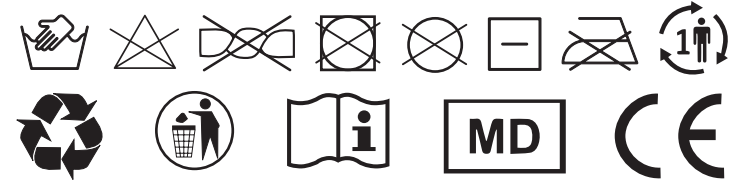
EN 1 The main body of the product is aluminum. The aluminum body is coated with a special material from the thermoplastic elastomer (TPE) class that is flexible and soft enough to suit the anatomy. Thanks to its ergonomic structure and design, it fits well on the area where it is applied.

2 The product is easily applied by passing the strap through the buckle. Thanks to this belt, the required pressure can be adjusted.

3 The terry cloth on the inner surface has a sweat absorbent structure.

4 Silicone pad which has optimum hardness provides comfortable support.

5 Can be applied to the right or left side.

6 It can be used for cases of lateral epicondylitis (tennis elbow) and medial epicondylitis (golfer's elbow). Whichever of these both problem the product will be used for, it is applied in such a way that the silicone pad part puts pressure on that area.

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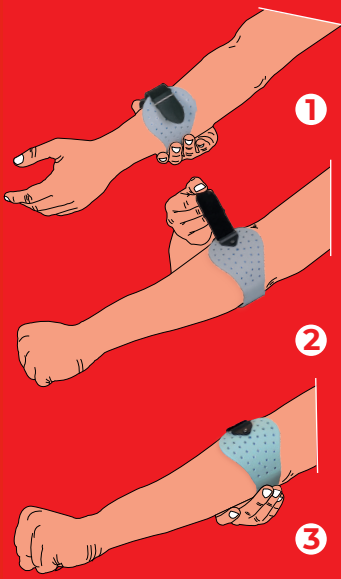
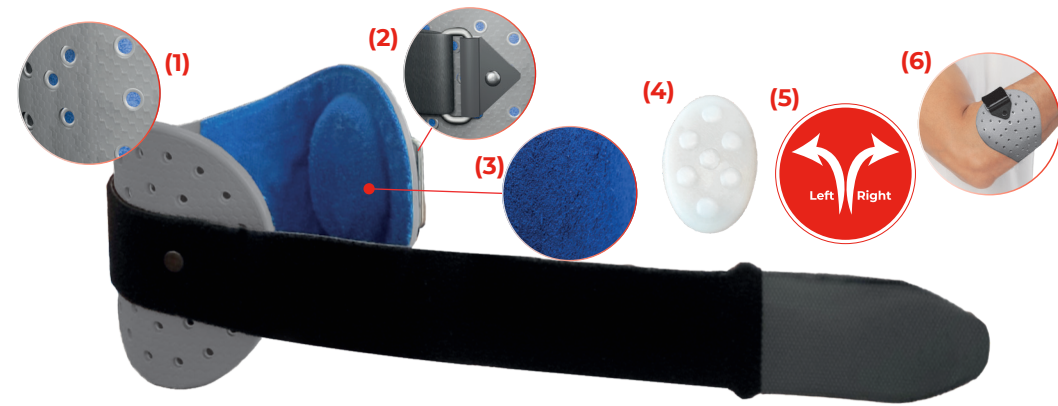
EN • In cases of lateral epicondylitis, the epicondylitis bandage is applied such a way that the silicone pad on the inside of the bandage comes to the lateral side.

• Extend your arm in a straight position. Apply the epicondylitis bandage from the lateral line of the elbow towards the wrist to the underside of the elbow, where your doctor indicates. If the bandage is applied in the right place, you will feel the pressure of the bandage during dorsiflexion movement. Do not apply more pressure than necessary.

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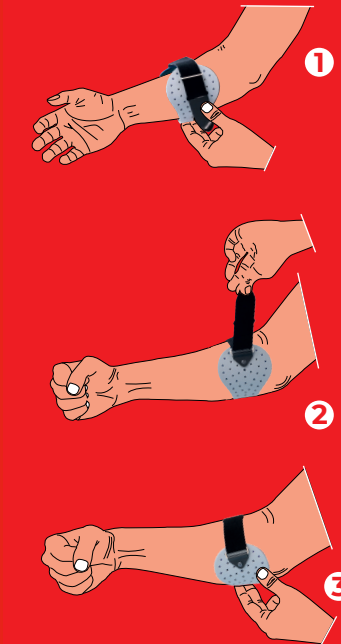
EN • In cases of medial epicondylitis, the epicondylitis bandage is applied such a way that the silicone pad on the inside of the bandage comes to the medial side.

• Extend your arm in a straight position. Apply the epicondylitis bandage from the medial line of the elbow towards the wrist, placing it just below the elbow as instructed by your doctor. If the bandage is applied correctly, you will feel the pressure it creates during flexion movement. Do not apply more pressure than necessary.

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